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www.muller-pharma.sk

Dr. Müller
PHARMA

MEDICAL DEVICE – INSTRUCTIONS FOR USE

Read instructions for use carefully before you start using vaginal suppositories.

Keep it, you may need to read it again.

Glycerine suppositories

Dr. Müller Pharma

Intended purpose:

Glycerine suppositories are medical devices intended to promote very gentle emptying of intestinal contents in thickened or hard stools, they locally irritate the rectal mucosa and also have lubricating effects during defecation at the same time.

Clinical benefit:

- Easiness of defecation

Mode of action:

The main intended function of the glycerine suppositories in the body is not obtained by pharmacological, immunological or metabolic means, but thanks to its physical properties, especially hygroscopicity and lubricant properties. The main component is glycerol, which promotes peristalsis and evacuation of the large bowel by virtue of its osmotic action, faecal softening actions and mild irritant effect. Character and its mechanism of action after rectal application is purely physical and nonspecific, it does not act by pharmacological, immunological or metabolic means.

The device is intended to be used by a lay person

- adults, pregnant and breastfeeding women and children aged 12 and up at variant 2.2 g
- children aged 1 to 12 years (under adult supervision) at variant 1.0 g

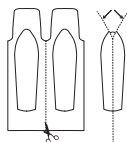
Indications:

- Acute constipation.
- Intermittent constipation.
- Habitual constipation.

How to use Glycerine suppositories:

Tear the foil above the pointed end of the suppository and remove the wrapper. Insert the suppository up into the rectum, pointed end first. The effect appears within 30 minutes after use.

A single glycerine suppository is intended for immediate single use. Re-application is neither possible nor feasible.

Glycerine suppositories 2.2 g – for adults and children aged 12 and up

Glycerine suppositories 1.0 g – for children

12 suppositories

Dosage:

Glycerine suppositories 2.2 g: Adults and children aged 12 and up 1 suppository daily.

Glycerine suppositories 1.0 g: Children aged 1 year and up 1 suppository daily.

Ingredients: Glycerin (min. 80 %), Sodium Stearate, Aqua.

Contraindications:

Acute abdominal pain or other pain of unknown origin.

Known hypersensitivity to any ingredient of the product.

Previously experienced inflammation or fissures in rectal area.

Bleeding haemorrhoids.

Side-effects:

Allergic reaction.

Irritation of rectal mucosa - mild rectal discomfort, a burning sensation, mild bleeding.

Warnings:

Glycerine suppositories are intended for rectal use only.

Concomitant usage with other rectal preparations consult with your doctor or pharmacist.

Use of suppositories in children up to one year of age consult with a doctor.

Do not use if the packaging is damaged.

Do not use after the expiry date which is stated on packaging.

Precautions:

Any serious adverse event that has occurred in relation to the use of device should be reported to the manufacturer and the national competent authority, and product should not be used again.

Storage:

Store at the temperature from +5 °C to +25 °C.

Keep out of reach of children.

Disposal of the device:

Dispose aluminium foil into mixed waste container and dispose of other parts of the packaging in the appropriate container for sorted waste (paper).

Date of issue of the latest revision of the instructions for use: 30.9.2024

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