Ty&Já®

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Určený účel použití:

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 2265
Dr. Müller PHARMA

MEDICAL DEVICE – INSTRUCTIONS FOR USE

Ty&Já®

LUBRICANT GEL silicone-based

Intended purpose:
Ty&Já® Lubricant gel silicone is medical device intended for lubrication of mucosa leading to:

- removing unpleasant feelings due to insufficient mucosal lubrication
- facilitation of temperature measurement or insertion of tampon into the vagina.

Clinical benefits:

- lubrication of the mucosa
- removal of the pain
- removing of discomfort
- improving the quality of sex
- improving the overall life quality.

Mode of action:
The main intended function of the gel in the body or on the surface is not obtained by pharmacological, immunological or metabolic means, but thanks to its physical properties, especially viscosity. It acts to increase the slipperiness of the surface of the vaginal epithelium and the skin. Character and its mechanism of action is purely physical, does not act by pharmacological, immunological or metabolic means.

Indications:

- Insufficient lubrication of mucosa during sexual intercourse.
- Pain and discomfort during sexual intercourse.
- Fever, menstruation.

The device is intended to be used by a lay person - adults, both men and women or to be used by healthcare professionals during temperature measurement.

It can be used with a condom.

How to use:
Remove the foil from tube, squeeze out sufficient amount of the gel, based on individual needs, and apply to already deployed condom or directly on the genital mucosa.

Ingredients: Dimethicone (100 %).

 **Dr. Müller Pharma s.r.o.,** U Mostku 182, 503 41 Hradec Králové, Czech Republic
info@muller-pharma.cz, tel.: +420 495 809 111
www.muller-pharma.com

SI 0253_001_00

 2265
Dr. Müller PHARMA

Read instructions for use carefully before you start using lubricant gel. Keep it, you may need to read it again.

50 ml tube

Contraindications:
Known hypersensitivity to any ingredient of the product.

Side-effects:
Allergic reaction
Decreased sperm motility

Warnings:
Lubricant gel is silicone-based. Concomitant usage with other vaginal (including vaginal contraception) or rectal preparations consult with your doctor or pharmacist.
If irritation occurs, stop using the lubricating gel. If irritation persists, consult your doctor.
If the application of the gel does not remove the discomfort from the lack of lubrication, consult your doctor.
It does not substitute contraception.
Do not use if the packaging is damaged.
Do not use after the expiry date listed on the package.

Precautions:
Use within 12 months after opening.
Any serious adverse event that has occurred in relation to the use of device should be reported to the manufacturer and the national competent authority and the product should not be used again in this case.

Storage:
Store at the temperatures from +5 °C to +25 °C.
Keep out of reach of children.

Disposal of the device:
Dispose of used tube into mixed waste container and dispose of other parts of the packaging in the appropriate container for sorted waste, according to the type of material (see recycling symbol).

Date of issue of the instructions for use: 30.9.2024

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