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Dátum vydania návodu na použitie:
11.10.2023

**Dr. Müller
PHARMA**

MEDICAL DEVICE - INSTRUCTIONS FOR USE

Intended purpose:

Ty&Já® lubricant gel is medical device intended for lubrication of mucosa leading to:

- removing unpleasant feelings due to insufficient mucosal lubrication
- facilitation of temperature measurement or insertion of tampon into the vagina.

Clinical benefits:

- lubrication of mucosa
- removing the pain
- removing unpleasant feelings
- improving quality of sex
- improving the overall life quality.

Mode of action:

The main intended function of the gel in the body or on the surface is not obtained by pharmacological, immunological or metabolic means, but thanks to its physical properties, especially viscosity and gel consistency. It acts to increase the slipperiness of the surface of the vaginal epithelium and the skin. Character and its mechanism of action is purely physical, does not act by pharmacological, immunological or metabolic means.

Indications:

- Insufficient lubrication of mucosa during sexual intercourse.

Read instructions for use carefully before you start using this device. Keep it, you may need to read it again.

- **NON-PERFUMED** • **PERFUMED WITH PEACH FLAVOUR**
- **PERFUMED WITH STRAWBERRY FLAVOUR**

50 ml tube, 100 ml dispenser

- Pain and unpleasant feelings during sexual intercourse.
 - Fever, menstruation.
- The device is intended to be used by a lay person - adults, both men and women or to be used by healthcare professionals during temperature measurement.
- It can be used with a condom.

How to use:

Tube: Remove the foil from tube, squeeze out sufficient amount of the gel, based on individual needs, and apply to already deployed condom or directly on the genital mucosa.

Dispenser: Take off the cap of the dispenser, squeeze out sufficient amount of the gel, based on individual needs, and apply to already deployed condom or directly on the genital mucosa.

Ingredients:

Aqua (min. 90 %), Glycerin (min. 2 %), PEG-8, Hydroxyethylcellulose (min. 1,5 %), Polysorbate 20, Aroma, Sodium Benzoate, Lactic Acid.
*Only perfumed variants contain aroma.

Contraindications:

Known hypersensitivity to any ingredient of the product.

Side-effects:

Allergic reaction.

Warnings:

Lubricant gel is water-based.

Concomitant usage with other vaginal preparations (including vaginal contraception) or rectal preparations consult with your doctor or pharmacist.

If irritation occurs, stop using the lubricating gel. If irritation persists, consult your doctor.

If the application of the gel does not remove the discomfort from the lack of lubrication, consult your doctor.

It does not substitute contraception.

Do not use if the packaging is damaged.

Do not use after the expiry date listed on the package.

Precautions:

Use within 6 months after opening.

Any serious adverse event that has occurred in relation to the use of device should be reported to the manufacturer and the national competent authority and the product should not be used again in this case.

Storage:

Store at the temperatures from +5 °C to +25 °C. Keep out of reach of children.

Disposal of the device:

Dispose of empty packaging in the appropriate container for sorted waste (paper, plastic).

Date of issue of the instructions for use:
11.10.2023

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