VšiVen[®] PLUS

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MEDICAL DEVICE - INSTRUCTIONS FOR USE

VšiVen[®] PLUS

FOR HAIR INFESTED WITH LICE AND NITS

Read instructions for use carefully before you start using this medical device. Keep it, you may need to read it again.

EN

120 ml

Intended purpose:

VšiVen[®] PLUS for hair infested with lice and nit is a medical device for topical application to the hair and scalp for the removal of Pediculus capitis and the prevention of the development of nits.

Clinical benefits:

- Removal of head lice and nits.
- Relief from itching.

Mode of action:

The main intended function of anti-head lice lotions on human scalp is not obtained by pharmacological, immunological or metabolic means, but thanks to its physical properties. It is recognized, that the mode of action is based on physical mechanisms and not on neurotoxicity. Predominant component of anti-head lice lotions is dimethicone. Exact dimethicone mode of action has been unclear, referred as asphyxiation or suffocation of lice due to blockage of the outer respiratory tract. However, there is evidence that occlusion of spiracles and tracheal trunks by silicone deposits produces a barrier that inhibits excretion of water, which results in osmotic stress leading ultimately to gut rupture. Tocopheryl acetate, a vitamin-E ester probably acts in a similar way to dimethicone. Therefore, anti-head lice lotions act via two physical mechanisms – blockage of the outer respiratory tract leading to suffocation of lice and inhibition of water excretion resulting in osmotic stress and gut rupture. Character and mechanism of action is exclusively physical, it does not act by pharmacological, immunological or metabolic effect.

Indications:

- Pediculosis.

The device is intended to be used by a lay person, adults, pregnant and breast-feeding women and/or children aged 6 months and up (under adult supervision).

How to use VšiVen[®] PLUS for hair infested with lice and nits?

1. Before starting the application, check the presence of lice in the hair of all family members. All persons with lice in their hair should be treated in one day to prevent the repeated transmission.
2. Cover the treated person's shoulders with a towel.
3. Unscrew the applicator cap. Apply the solution on the hair evenly from the forehead to the top of the head, and from the top of the head to the ears and the nape of the neck. The amount required depends on the hair type and length. For shoulder-length hair, about 30-50 ml of the product is required.
4. Apply sufficient amount of the product on the hair and the scalp. Rub it in from the hair roots to the ends. Let it act for 15 minutes. In case of stronger infestation, the product may act for up to 30 minutes.
5. Wash the product with your common shampoo and wash the hair thoroughly with warm water. Hair conditioner is recommended for longer hair.
6. Comb the hair by means of a common comb and divide it to 1-cm wide strands. Comb the individual strands repeatedly by the metal comb provided to remove the dead lice and nits. Start the hair combing as close to the hair roots as possible. Separate the strands freed from nits from the still not combed hair.
7. Wash the comb with warm water and soap after every use. We recommend repeating the application according to the above procedure after 7-9 days to prevent the hatching of new lice from unaffected unremoved nits.
8. Wash all clothes, blankets and towels used at the temperature of 60 °C. Disinfect all combs and hair brushes etc. Vacuum or wipe the floor. Remove the bag from the vacuum cleaner.

Ingredients: Dimethicone, Tocopheryl Acetate, Prunus Amygdalus Dulcis Oil, Prunus Armeniaca Kernel Oil.

Contraindications:

Children up to 6 months.

Known hypersensitivity to any ingredient of the product. Open or unhealed wounds on the scalp or otherwise damaged scalp skin.

Side-effects:

Allergic reaction

Warnings:

For external use only.

In case of the product ingestion, seek medical help immediately.

If crawling lice are still seen after a full course of treatment contact your health care provider.

Contact with eyes should be avoided. If, however, the product gets to eyes, flush them thoroughly by cold water.

Precautions:

Do not use after the lapse of the shelf life marked on the pack.

Any serious adverse event that has occurred in relation to the use of device should be reported to the manufacturer and the National competent authority, and product should not be used again.

Storage:

Keep out of reach of children.

Store at the temperature of from +5 °C to +25 °C.

Cover the bottle when you're done using it and put it in the original box with the instructions for use.

Content of the pack: Solution in an applicator bottle + metal comb.

Disposal of the device:

Dispose of used bottle into mixed waste container and dispose of other parts of the packaging in the appropriate container for sorted waste, according to the type of material (see recycling symbol).

Date of issue of the latest revision

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