# Hyalosan

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Dátum vydania návodu na použitie: 11.10.2023

## MEDICAL DEVICE – INSTRUCTIONS FOR USE

# Hyalosan lubricant gel with hyaluronic acid

#### Intended purpose:

**HYALOSAN lubricant gel with hyaluronic acid** is medical device intended for lubrication of mucosa leading to:

- removing unpleasant feelings due to insufficient mucosal lubrication
- facilitation of temperature measurement or insertion of tampon into the vagina.

#### Clinical benefits:

- lubrication of mucosa
- removing the pain
- removing unpleasant feelings
- improving quality of sex
- improving the overall life quality.

#### Mode of action:

The main intended function of the gel in the body or on the surface is not obtained by pharmacological, immunological or metabolic means, but thanks to its physical properties, especially viscosity and gel consistency. It acts to increase the slipperiness of the surface of the vaginal epithelium and the skin. Character and its mechanism of action is purely physical, does not act by pharmacological, immunological or metabolic means.

#### Indication:

- Insufficient lubrication of mucosa during sexual intercourse.
- Pain and unpleasant feelings during sexual intercourse.
- Fever, menstruation.

The device is intended to be used by a lay person - adults, both men and women or to be used by healthcare professionals during temperature measurement.

It can be used with a condom.

#### How to use:

**Tube:** Remove the foil from tube, squeeze out sufficient amount of the gel, based on individual needs, and apply to already deployed condom or directly on the genital mucosa.

**Dispenser:** Take off the cap of the dispenser, squeeze out sufficient amount of the gel, based on individual needs, and apply to already deployed condom or directly on the genital mucosa.

Read instructions for use carefully before you start using lubricant gel. Keep it, you may need to read it again.

50 ml tube, 50 ml dispenser

#### Ingredients:

Sodium Hyaluronate (min. 0,05 %), Glycerin (min. 3 %), Propylene Glycol (min. 4 %), Benzoic Acid, Hydroxyethylcellulose (min. 1,5 %), Aqua (min. 85 %).

#### Contraindications:

Known hypersensitivity to any ingredient of the product.

#### Side-effects:

Allergic reaction.

#### Warnings:

Lubricant gel with hyaluronic acid is water-based. Concomitant usage with other vaginal preparations (including vaginal contraception) preparations consult with your doctor or pharmacist.

If irritation occurs, stop using the lubricating gel. If irritation persists, consult your doctor.

If the application of the gel does not remove the discomfort from the lack of lubrication, consult your doctor.

It does not substitute contraception.

Do not use if the packaging is damaged.

Do not use after the expiry date listed on the package.

#### Precautions:

Use within 3 months after opening.

Any serious adverse event that has occurred in relation to the use of device should be reported to the manufacturer and the national competent authority, and product should not be used again in this case.

#### Storage:

Keep out of reach of children.

Store at the temperatures from +5 °C to +25 °C.

#### Disposal of the device:

Dispose of empty packaging in the appropriate container for sorted waste (paper, plastic).

Date of issue of the instructions for use: 11.10.2023

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